

# iScreen™ Mono



## Mononucleosis Rapid Test Cassette

*Simple, point-of-care detection of Infectious Mononucleosis*

- CLIA Waived whole blood procedure
- On board procedural control
- Room temperature storage
- Results in minutes

Brands You Can Trust™



# iScreen™ Mono

## Mononucleosis Rapid Test Cassette

Simple, point-of-care detection of Infectious Mononucleosis



### Kit includes:

- External positive/negative controls
- Developer buffer
- Transfer pipette



### Features

Screens whole blood,  
serum & plasma samples.....  
CLIA Waived for  
whole blood samples.....  
Results in minutes.....  
Room temperature storage.....  
Very high overall accuracy.....

### Benefits

Flexible testing  
Simple laboratory procedure  
Rapid results for early diagnosis  
Convenient handling  
Confidence in test results

<b>Product Code:</b>	<b>Kit Size:</b>	<b>Description:</b>	<b>Complexity:</b>
I-MO-402.....	20/BX.....	Cassette.....	CLIA Waived/whole blood, Moderate/serum, plasma

iScreen™ Mono positive result shown above.

## Instructions for I-MO-402

### Procedure


Read procedure before running test. Refer to package insert for details.

Place test device on a clean, dry, level surface.

**Venipuncture Whole Blood**  
Add 2 Drops of Blood  
(about 50 µL), then 1 Drop  
of Buffer to the Sample Well.

**Fingerstick Whole Blood**  
Add 1 Capillary Tube of Blood  
(about 50 µL), then 1 Drop  
of Buffer to the Sample Well.

**Serum/Plasma**  
Add 1 Drop of Serum or Plasma  
(about 25 µL), then 1 Drop  
of Buffer to the Sample Well.

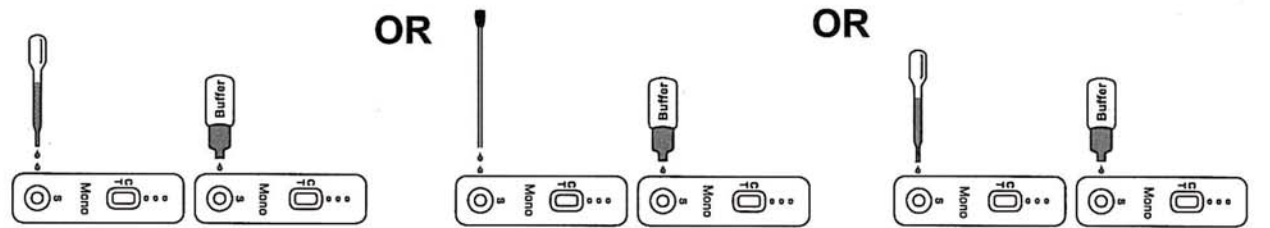

### Interpretation of Results

Read result at five (5) minutes. Do not move test device until ready to read.

